



Preparing for Sudden Cardiac Arrest

## AED Trainer 3

*The AED Trainer 3 is the ultimate preparation tool for trained responders facing cardiac arrest.*



# AED Trainer 3

A fast, accurate response from trained responders when treating the cardiac arrest patient is essential if successful patient outcomes are to be achieved.

## Looks and works like a HeartStart FR3

The AED Trainer 3 reflects the unique functionality of the Philips HeartStart FR3 AED, and provides a realistic training experience for emergency responders to deliver optimal therapy when facing cardiac arrest.

No actual defibrillation shocks are delivered by the device assuring safety throughout each training session.

## Flexible Configuration

The AED Trainer 3 is configured with eight real-world scenarios that have been developed in accordance with internationally recognized responder programs. They are AHA and ECC 2010 Guidelines compliant. Many parameters on each scenario can be easily configured to conform to local protocols or future Guidelines updates. Language version for voice prompts can also be changed at a touch of a button.

## Complete Instructor Control

The instructor can change scenarios by pressing the buttons on the device itself or by using an optional remote control. In addition, the remote control lets the instructor pause, change or introduce new challenges during each scenario to test how students respond to a variety of situations, enhancing the learning experience.



### Practice Fast Delivery of Therapy

The 'auto-on' carry case and pre-connected training pads\* enables students to practice fast delivery of therapy just like on the HeartStart FR3.




Shown with Remote Control, not included with AED Trainer 3



### Laerdal LINK Technology Enabled

Indicates correct/incorrect placement of pads via LED's on Laerdal LINK enabled manikins.

## Simple to Configure

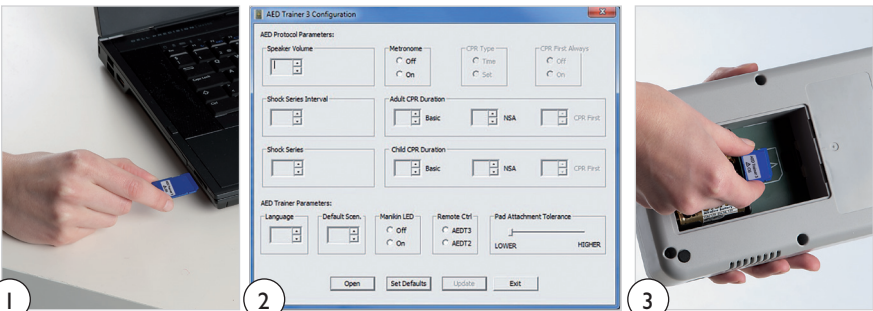

1 Insert the Operating System Card into a PC

2 Change the parameters


3 Return to AED Trainer 3

## Training Infant/Child Mode

The infant/child key removes the need to modify settings and facilitates the practice of prompt pediatric defibrillation on infants or children over 8 years of age.



Training Infant/Child key

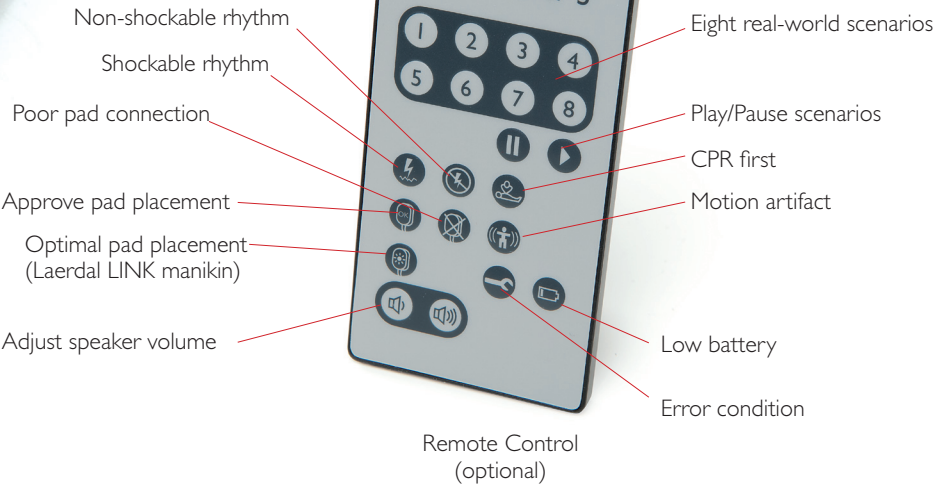









Remote Control (optional)

Visit [www.laerdal.com/downloads](http://www.laerdal.com/downloads) for multilanguage DFU, latest localizations and configuration software

\* The AED Trainer 3 can be used with HeartStart Training Pads III or Laerdal LINK Pads.

## AED Trainer 3 Scenarios:

Scenario	Symbol	Description
1	 One shock for conversion	<ul style="list-style-type: none"> <li>Shockable rhythm</li> <li>One shock needed for conversion</li> <li>Non-shockable rhythm</li> </ul>
2	 One shock for conversion	<ul style="list-style-type: none"> <li>Non-shockable rhythm</li> <li>Shockable rhythm</li> <li>One shock needed for conversion</li> <li>Non-shockable rhythm</li> </ul>
3	 Troubleshooting pads, one shock for conversion	<ul style="list-style-type: none"> <li>Poor pads contact</li> <li>Shockable rhythm</li> <li>One shock needed for conversion</li> <li>Non-shockable rhythm</li> </ul>
4	 Refribrillation after conversion	<ul style="list-style-type: none"> <li>Shockable rhythm</li> <li>One shock needed for conversion</li> <li>Non-shockable rhythm</li> <li>Return to shockable rhythm after three minutes</li> <li>One shock needed for conversion</li> <li>Non-shockable rhythm</li> </ul>
5	 Non-shockable rhythm	<ul style="list-style-type: none"> <li>Non-shockable rhythm throughout</li> </ul>
6	 Two shocks for conversion	<ul style="list-style-type: none"> <li>Shockable rhythm</li> <li>Two shocks needed for conversion</li> <li>Non-shockable rhythm</li> </ul>
7	 CPR first, one shock for conversion	<ul style="list-style-type: none"> <li>Shockable rhythm</li> <li>CPR First</li> <li>Shockable rhythm</li> <li>One shock needed for conversion</li> <li>Non-shockable rhythm</li> </ul>
8	 Shockable rhythm	<ul style="list-style-type: none"> <li>Shockable rhythm throughout</li> </ul>

\* The AED Trainer 3 can be used with HeartStart Training Pads III or Laerdal LINK Pads.

## ORDERING INFORMATION

### Part No.

198-00150

### Description

AED Trainer 3, DFU, Quick Start Guide, 4 AA Alkaline Batteries, Training Infant/Child Key with Tether, Carry Case, Operating System Card, HeartStart Training Pads III, HeartStart Interconnect Cable, External Manikin Adapter.

### Consumables

#### Part No.

989803150181

#### Description

HeartStart Training Pads III

989803150201

HeartStart Training Pads III

Interconnect Cable

945090

Laerdal LINK Training Pads

### Accessories

#### Part No.

198-00350

#### Description

Remote Control (with CR2025 3V Battery)

198-10050


Hard Carry Case

198-10250

IGB Operating System Card w/Software

M5089A

External Manikin Adapters (5 pack)

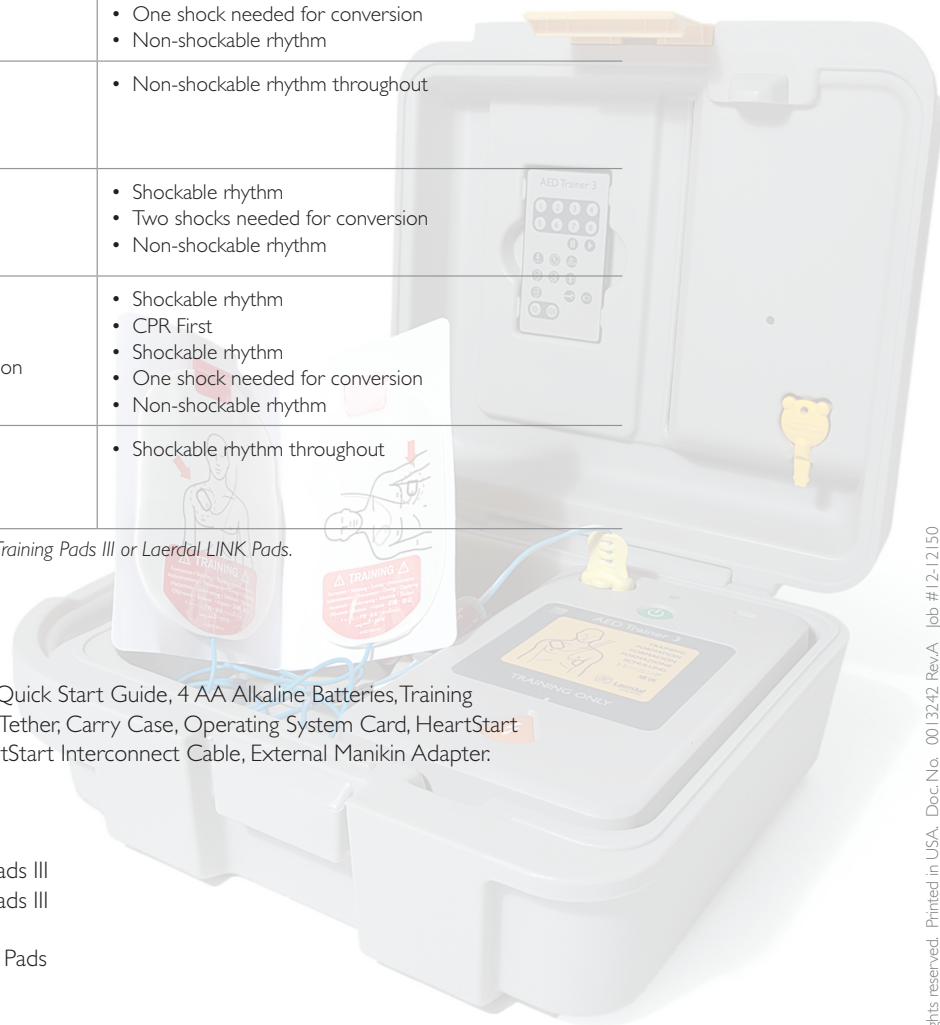

**Laerdal®**  
helping save lives