

Sounds Trainer



ENGLISH

Directions for Use



Table of Contents

Laerdal Recommends..... 1

Items Included..... 1

Skills Taught..... 1

Auscultation of Heart, Breath & Bowel Sounds.....2

The SimPad Sounds Trainer is an accessory to the SimPad System that enables the auscultation and recognition of normal and abnormal heart, breath and bowel sounds.

Laerdal Recommends

For use with standard stethoscope.

Items Included:

- (4) SimPad Sounds Trainer External Speaker
- (1) Connector Cable
- (1) Carry Case

Skills Taught:

- Auscultation of Normal and Abnormal Heart, Breath & Bowel Sounds.

Sounds Include:

See SimPad DFU for complete list of available sounds.

Auscultation of Heart, Breath and Bowel Sounds-

1. Secure connector cable to back of SimPad Link Box. (Photo 1)
If necessary, use the cable adapter provided with SimPad

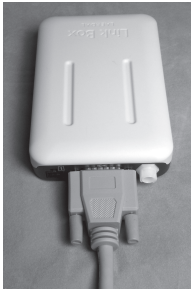


Photo 1

2. Attach each external speaker to the connector cable via attached cord.
(Photo 2)

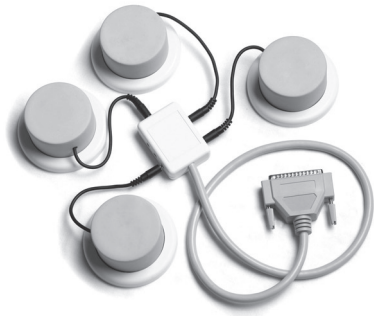


Photo 2

3. Auscultate sounds by placing a stethoscope against the top surface of an external speaker

See SimPad Directions for Use for complete instructions and operation of the Heart, Breath and bowel Sound features.