


Newborn Mask

Snap Design

User Guide

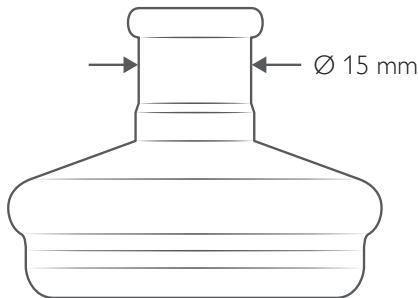

846156 - Size 0, 10 pack

846157 - Size 1, 10 pack

www.laerdalglobalhealth.com



Laerdal

helping save lives

Newborn Masks are intended for face mask ventilation of newborns and infants.

This User Guide covers both size 0 and size 1 models.

 Caution

The mask may be reused provided proper cleaning and sterilization/disinfection procedures are followed between each patient use. It must be cleaned and sterilized/disinfected before first patient use.

The mask fits with standard 15 mm inner-diameter conical connectors, as defined by ISO 5356-1. Check fit before use with your 15 mm connector. When used with Upright Newborn Bag-Mask the mask attaches with a snap fit when pressed completely into place.

Reprocessing Instructions

1. Cleaning

1. Wash in a clean tray with clean water and mild soap.
2. Use a scrub or brush to remove any soil.
3. Rinse in clean water to remove all soil and soap.
4. Repeat above steps until clean.

2. Sterilization or Disinfection

Sterilize or disinfect the mask by one of these methods:

- Sterilization - Steam Autoclaving
136 °C (276 °F), 2.0 kg/cm², 10-20 minutes
- Disinfection - Boiling in clean water*
100 °C (212 °F), 10 minutes.

* Validated at approximately sea-level pressure.

- Disinfection - Chemical
 1. Activated glutaraldehyde solution: Use the disinfectant manufacturer's recommended procedure.
 2. Immerse for 60 minutes.
 3. Remove parts using aseptic technique.
 4. Rinse 3 times, for 1 minute using fresh clean water each time.

3. Dry and Inspect

1. Allow the mask to air dry.
2. Visually inspect for damage and cleanliness/mineral deposits.
3. Remove damaged or unclean masks from service.

Regulatory Information

Symbol	Definition
CE 2460	This product is in compliance with the essential requirements of EU Council directive 93/42/EEC as amended by EU Council directive 2007/47/EC.

Technical Information	
Material	Silicone rubber
Mass	17 g (size 0), 37 g (size 1)
Outer diameter	49 mm (size 0), 64 mm (size 1)
Operating environmental conditions	-18 °C to 50 °C (-0.4 °F to 122 °F)
Storage environmental conditions	-40 °C to 60 °C (-40 °F to 140 °F)
Newborn Mask - Snap design is designed to fit with 15 mm inner diameter female conical connectors, but deviates from ISO 5356-1:2004 Anaesthetic and respiratory equipment - Conical connectors - Part 1: Cones and sockets.	

Laerdal Global Warranty

Refer to the Laerdal Global Warranty for terms and conditions. For more information visit www.laerdal.com

© 2017 Laerdal Medical AS, All rights reserved.

Pending US and international design registration.
Laerdal® is a registered trademark of Laerdal Medical AS.

Manufactured in China for:

Laerdal Medical AS

P.O. Box 377, Tanke Svilandsgate 30, 4002 Stavanger, Norway

Tel : +47 51 51 17 00

Fax: +47 51 52 35 57

Email: laerdal.norway@laerdal.no

Printed in China

20-08756 Rev C

www.laerdalglobalhealth.com



Laerdal
helping save lives